MEDRAD® Twist & Go

Sterile High Pressure Connector Tubing Products
Produits de tubulure de raccord haute pression stérile
Tubo conector de alta presión estéril

Instructions for Use / Mode d'emploi / Instrucciones de Uso

Catalog Number Référence catalogue Número de Catálogo	Product Name Nom du Produit Nombre del Producto
TAG C50 HPCT	Sterile High Pressure Connector Tubing, 50 cm (20 in) Tubulure de raccord haute pression stérile, 50 cm Tubo conector de alta presión estéril de 50 cm (20")
TAG C75 HPCT	Sterile High Pressure Connector Tubing, 75 cm (30 in) Tubulure de raccord haute pression stérile, 75 cm Tubo conector de alta presión estéril de 75 cm (30")
TAG C120 HPCT	Sterile High Pressure Connector Tubing, 120 cm (48 in) Tubulure de raccord haute pression stérile, 120 cm Tubo conector de alta presión estéril de 120 cm (48")
TAG C150 HPCT	Sterile High Pressure Connector Tubing, 150 cm (60 in) Tubulure de raccord haute pression stérile, 150 cm Tubo conector de alta presión estéril de 150 cm (60")

©2013-2014, 2017, 2019 Bayer. This material may not be reproduced, displayed, modified or distributed without the express prior written consent of Bayer.

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Bayer, the Bayer Cross, MEDRAD, MEDRAD Twist & Go, and Twist & Go are trademarks owned by and/or registered to Bayer in the U.S. and/or other countries.

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Vincent Medical

(Dong Guan) Mfg. Co., Ltd.

45-46 ShaBu Industrial Zone
Qiao Long District

Tang Xia Town,
Dong Guan City
Guangdong, China 523730

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A glossary of the symbols used on the MEDRAD® Twist & Go sterile high pressure connector tubing products can be found on page 2 of these Instructions for Use.

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Introduction: Read the information contained in these instructions for use. Understanding the information will assist you in using the **MEDRAD® Twist & Go** (Twist & Go) High Pressure Connector Tubing properly.

Important Safety Notice: The **Twist & Go** High Pressure Connector Tubing is intended to be used by qualified medical professionals with proper training and experience in angiographic procedures and the use of injectors from Bayer. As with any injector, operator vigilance is required when using an Injection System. Visual inspections and good practice during set-up and use and compliance with the operation manual remain the primary methods to limit the risk of air being present during fluid delivery.

Indications for Use: The **Twist & Go** High Pressure Connector Tubing is intended to be used in the delivery of contrast media and common flushing solutions. The device is indicated for single use only with injectors from Bayer with **Twist & Go** Syringes.

Contraindications: None known.

Restricted Sale: Rx Only.

Report any serious incident that has occurred in relation to this device to Bayer (radiology.bayer.com/contact) and to your local European competent authority (or, where applicable, to the appropriate regulatory authority of the country in which the incident has occurred).

⚠ WARNING

Air Embolism Hazard. Serious patient injury or death may result.

- Ensure that one operator is designated as being responsible for purging the High Pressure Connector Tubing. Do not change operators during the procedure. If an operator change must occur, ensure that the new operator verifies that the fluid path is purged of air.
- Ensure that patient is not connected while purging air from High Pressure Connector Tubing.
- Remove all air from the High Pressure Connector Tubing.
- Use caution when removing air from the syringe or tubing connections. Component damage may result from the use of tools during air removal.
- Securely tighten the high pressure connector tubing to syringe tip connection. Component damage may result from the use of tools during tightening.
- Do not use tubing to manipulate the injector position.
- Avoid contact of and damage to syringe tip, tubing and connections when manipulating injector position.

⚠ WARNING

Bloodborne Contamination Hazard. Serious patient and/or worker injury or death may result.

- Use caution when removing air from the syringe or tubing connections. Component damage may result from the use of tools during air removal.
- Do not use tools to securely tighten the high pressure connector tubing to syringe tip connection.
- Avoid contact of and damage to syringe tip, tubing and connections when manipulating injector position.
- For devices labeled for single use, please note: This product is intended for single use only. Do not resterilize, reprocess, or reuse. The disposable devices have been designed and validated for single use only. Re-use of the single use disposable devices pose risks of device failure and risks to the patient. Potential device failure includes significant component deterioration with extended use, component malfunction, and system failure. Potential risks to the patient include injury due to device malfunction or infection as the device has not been validated to be cleaned or re-sterilized.
- Do not exceed 1200 psi (8273 kPa).
- Do not use tubing to manipulate injector position.

Biological Contamination Hazard. Serious patient and/or worker injury or death may result.

- Properly discard disposables after use if contamination may have occurred during setup or use.

⚠ CAUTION

Environmental Contamination Hazard. Patient and/or worker injury may result.

- Do not use if sterile package is opened or damaged. Patient or operator injury may result if package is opened or damaged, or if damaged components are used. Visually inspect contents and package before each use.
- Follow sterile technique principles, specifically, maintain sterility of the syringe tip and high pressure connector tubing.

INSTALLING, FILLING AND PURGING A HIGH PRESSURE CONNECTOR TUBING

1. Remove the dust cap from the syringe tip if attached.
2. Insert the **Twist & Go** High Pressure Connector Tubing onto the **Twist & Go** syringe tip.
3. Turn the **Twist & Go** (A) nut clockwise to securely tighten the high pressure connector tubing to the syringe tip (B).
4. Purge all air from the high pressure connector tubing. Use the injector to push contrast and flushing solutions out until all of the air has been removed from the High Pressure Connector Tubing. Gentle tapping at any connection point may be needed to facilitate air removal.

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