# Workflow Hub for MEDRAD® Centargo CT Injection System

Addendum to Operation Manual

# Languages

|                           |     |
|---------------------------|-----|
| ENGLISH .....             | 1   |
| ČESKY .....               | 7   |
| DANSK .....               | 13  |
| DEUTSCH .....             | 19  |
| EESTI .....               | 25  |
| ESPAÑOL (ESPAÑA).....     | 31  |
| FRANÇAIS .....            | 37  |
| HRVATSKI .....            | 43  |
| INDONESIA .....           | 49  |
| ITALIANO .....            | 55  |
| LATVIEŠU .....            | 61  |
| LIETUVIŲ .....            | 67  |
| MAGYAR .....              | 73  |
| NEDERLANDS .....          | 79  |
| NORSK .....               | 85  |
| POLSKI .....              | 91  |
| PORTUGUÊS (EUROPEU) ..... | 97  |
| ROMÂNĂ .....              | 103 |
| SLOVENŠČINA .....         | 109 |
| SRPSKI .....              | 115 |
| SLOVENSKY .....           | 121 |
| SUOMI .....               | 127 |
| SVENSKA .....             | 133 |
| TÜRKÇE .....              | 139 |
| TIẾNG VIỆT .....          | 145 |
| ΕΛΛΗΝΙΚΑ .....            | 151 |
| БЪЛГАРСКИ .....           | 157 |
| ภาษาไทย .....             | 163 |
| 한국어 .....                 | 169 |
| 日本語 .....                 | 175 |

# Workflow Hub for MEDRAD® Centargo CT Injection System

## Addendum to Operation Manual

Report any serious incident that has occurred in relation to this device to Bayer ([radiology.bayer.com/contact](http://radiology.bayer.com/contact)) and to your local European competent authority (or, where applicable, to the appropriate regulatory authority of the country in which the incident has occurred).

|    |     |
|----|-----|
| EC | REP |
|----|-----|

**Bayer Medical Care B.V.**

Avenue Céramique 27  
6221 KV Maastricht  
The Netherlands

|    |     |
|----|-----|
| CH | REP |
|----|-----|

**Bayer (Schweiz) AG**

Uetlibergstrasse 132  
8045 Zurich

|    |     |
|----|-----|
| UK | REP |
|----|-----|

**Bayer Plc**

400 South Oak Way  
Green Park  
Reading  
Berkshire  
RG2 6AD  
United Kingdom

# 1 Using Workflow Hub with MEDRAD® Centargo CT Injection System

This document is an addendum to the MEDRAD® Centargo Operation Manual and provides specific information for configuring and using the Workflow Hub, Catalog number CENT-HUB. Refer to the MEDRAD® Centargo Operation Manual for additional information.

## 1.1 Workflow Hub

The Workflow Hub is an updated version of the Control Room Unit that replaces the version currently shown in the Centargo operation manual. The Workflow Hub has the same functionality as the original version of the Control Room Unit.

**NOTE:** The Workflow Hub is not for use in the scan room.

**NOTE:** The power cord is the mains (AC power) disconnection device that isolates and removes AC power from the Workflow Hub. Do not place the Workflow Hub in front of AC wall outlets and prevent access to the supply mains.

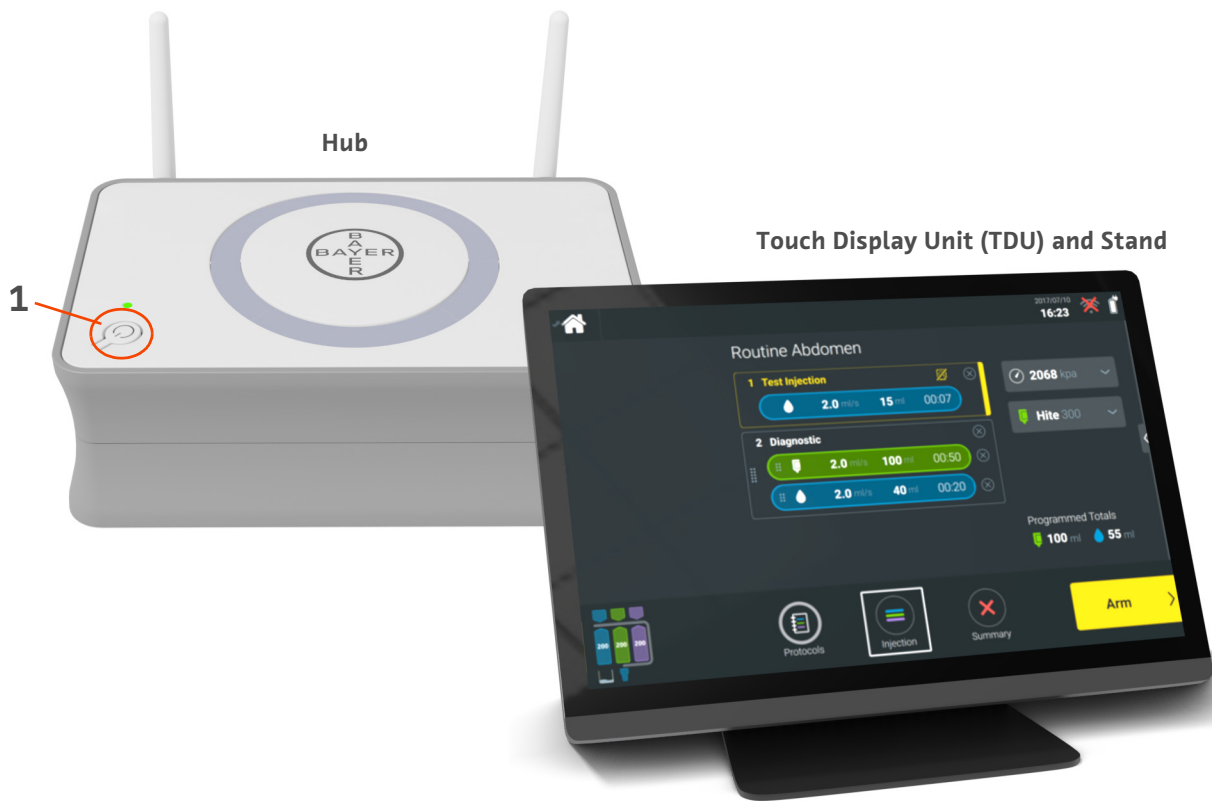
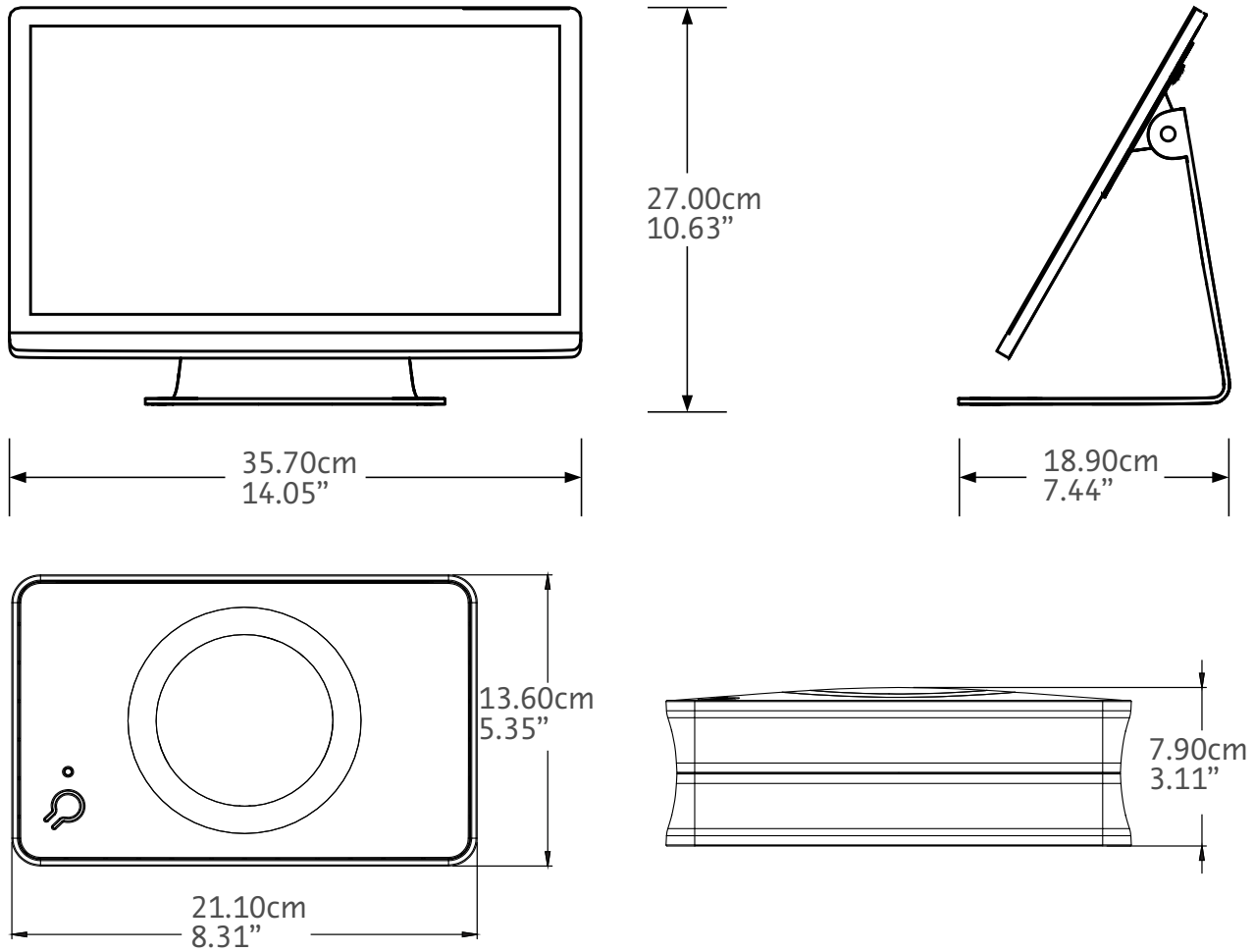
Figure 1 - 1: Workflow Hub

| # | Icon  | Name         |
|---|---|--------------|
| 1 |  | Power On/Off |

## 1.2 Workflow Hub Specifications

### 1.2.1 Workflow Hub Dimensions and Weight

**NOTE:** Listed weight and dimensions are approximate.

Weight:

Hub: 1.2 kg (2.7 lbs)

TDU: 3.2 kg (6.61 lbs)

### 1.2.2 Workflow Hub Power Requirements

100-240 VAC

50-60 Hz

1.3A

### 1.2.3 Workflow Hub Symbols and Connections

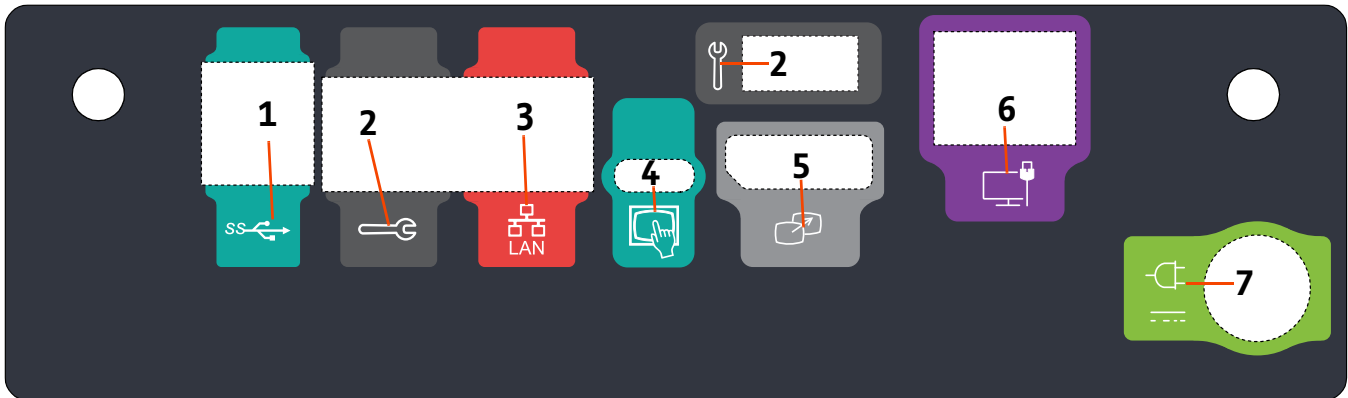
Figure 1 - 2: Workflow Hub, Rear Port View

| # | Icon | Function                                      | Description  |
|---|------|---|--|
| 1 |      | Identifies USB connections.                   | USB connections.   |
| 2 |      | Identifies service port.                      | Bayer use only. This connection is for Bayer service use or for Connect.CT scanner connectivity  |
| 3 |      | Identifies computer network connection.       | Use to connect the system to the local area network (RIS).   |
| 4 |      | Identifies connection for Touch Display Unit  | Use to connect the touch screen display to the hub.  |
| 5 |      | Identifies connection for secondary display   | Bayer use only. This connection for screen extension or transfer to a second display.  |
| 6 |      | Identifies a network connection.              | Use for wired Workflow Hub connection to the Injector.   |
| 7 |      | Identifies power input and supply connection. | Power input and supply connection.<br><br><b>NOTE:</b> For the power input and supply connection, Workflow Hub requires use of a Listed Class 2, LPS, or Listed Information Technology Equipment (ITE) power supply with output rated 12Vdc, minimum 6A, marked LPS. Use only part number from Bayer for power supply. |

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2022-05-31

Bayer reserves the right to modify the specifications and features described herein or to discontinue any product or service identified in this publication at any time without prior notice or obligation. Please contact your authorized Bayer representative for the most current information.

The patient data that appear in this document are fictitious protected health information (PHI) and/or actual PHI from which all personally identifiable information (PII) have been removed or otherwise anonymized. No personally identifiable information is show.

Bayer, the Bayer Cross, MEDRAD, Centargo and MEDRAD Centargo, are trademarks owned by and/or registered to Bayer in the U.S. and/or other countries. Other trademarks and company names mentioned herein are properties of their respective owners and are used herein solely for informational purposes. No relationship or endorsement should be inferred or implied.

© 2021-2022 Bayer. This material may not be reproduced, displayed, modified, or distributed without the express prior written consent of Bayer.

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To provide feedback or request support, please use the contact form provided on [radiology.bayer.com/contact](http://radiology.bayer.com/contact)

**Manufacturer**  
Imaxeon Pty Ltd  
Unit 1, 38-46 South Street  
Rydalmere NSW 2116  
Australia  
Phone: +61 2 8845 4999  
Fax: +61 2 8845 4998

625 Alpha Drive  
Pittsburgh, PA 15238  
U.S.A.